

**NOVA-ONE<sup>®</sup>**  
**DIAGNOSTICS**  
**A1C CONTROL LEVELS 1 & 2**

**P/N: NOD13122-100 (4 Vial) & NOD13111-100 (2 Vial)**

<b>Product</b>	<b>Lot Number</b>	<b>Expiration Date</b>
<b>NOD<sup>®</sup> A1c Control, Level 1&amp; 2</b>	<b>3220N001</b>	<b>April 2012</b>

**INTENDED USE**

NOD<sup>®</sup> A1c Control is intended for use as quality control material to monitor the performance and precision of Hemoglobin A1c (HbA1c %) HPLC and Immunoassay test methods using protocols established in individual laboratories.

**SUMMARY AND PRINCIPLE**

Laboratories should run quality control materials with all patient samples to ensure proper performance of instruments and reagents. NOD<sup>®</sup> A1c Control is provided at two levels – normal (L1) and abnormal (L2).

**REAGENT**

NOD<sup>®</sup> A1c Control is prepared from human whole blood to which stabilizers are added. The product is provided in liquid form for user convenience. No further dilution of the control solution is needed.

**STORAGE AND STABILITY**

Kits may be received thawed but should be stored frozen (- 15°C to - 25°C) immediately upon receipt. Unopened NOD<sup>®</sup> A1c Control is stable until the expiration date printed on the container when stored frozen. Refrigerated vials are stable for 180 when stored properly and tightly capped.

*(MinuteClinic Sites Please Note: MinuteClinic does not freeze control materials. The controls should be refrigerated and dated immediately upon receipt and will expire 180 days from the date received when stored properly and tightly capped.)*

Upon receiving the kit, mark the specific date refrigerated storage began on vial; product is stable for 180 days when stored at 2-8°C in **tightly** closed containers. Aliquots made immediately from freshly open vials may be frozen one time and stored until expiration date printed on the container. Thawed aliquots cannot be refrozen. Dried red control residue in top of cap means vials are not being recapped properly. (Please Note) Refrigerated Stability for HPLC methods is 120 days.

**PROCEDURE**

NOD<sup>®</sup> A1c Control should be treated in the same manner as patient samples in accordance with instructions for testing determination method being used. No further dilution of the control solution is needed. Frozen control should be thawed at 2-8°C and mixed by gentle inversion several times prior to use. Refrigerated control should be mixed by gentle inversion prior to use Do not shake vigorously. Always clean any excess control material from dropper tip before tightly recapping a vial.

**LIMITATIONS**

Different values from those obtained with reagents available at the time of assay may be obtained as a result of changes in manufacturer's reagents or lot-to-lot reagent variability. NOD<sup>®</sup> A1c

**NOVA-ONE Diagnostics<sup>®</sup>, LLC 22287 Mulholland Hwy Calabasas, CA. 91302**  
**818-348-1543 Fax 818-348-9696**  
**Email [nova-one@sbcglobal.net](mailto:nova-one@sbcglobal.net) / [www.nova-one.com](http://www.nova-one.com)**  
**(REV 06-01-2010)**

**NOVA-ONE<sup>®</sup>**  
**DIAGNOSTICS**  
**A1C CONTROL LEVELS 1 & 2**

**P/N: NOD13122-100 (4 Vial) & NOD13111-100 (2 Vial)**

Control should not be used past its expiration date or after improper handling. Microbial contamination will affect performance of this product.

**ANALYTE VALUES**

In accordance with good laboratory practices, each laboratory should establish its own analyte means and acceptable performance ranges.

**SPECIFIC PERFORMANCE CHARACTERISTICS**

NOD<sup>®</sup> A1c Control is manufactured in accordance with industry guidelines and standards. To perform as intended, the control requires proper storage and handling as described in this package insert.

**WARNING**

Biological source material, treat as potentially infectious. All human plasma units used in manufacturing this product have been tested according to by FDA accepted methods and found non-reactive or negative for Hepatitis B Surface Antigen (HbsAg), HCV antibodies, and HIV-1/2 antibodies. This product may contain other human or animal source materials for which there are no approved tests and should be considered as potentially infectious for Hepatitis B (HBV), Hepatitis C (HCV), HIV-1, HIV-2, HTLV-I, HTLV-II, as well as any other infectious agents, and handled with the same precautions used in handling patient specimens.

**Assigned Values and Ranges Lot #3220N001 (Representative Values)**  
**(Containing Vial Lots L1 #3222N001 & L2 #3224N001)**

For In Vitro Diagnostic Use	UNITS	LEVEL 1 – 3222N001			LEVEL 2 – 3224N001		
		MEAN	Expected RANGE		MEAN	Expected RANGE	
BAYER A1cNOW Plus+	%	5.5	4.4	6.6	8.6	6.9	10.3
Siemens DCA 2000 “Vantage”	%	5.6	4.5	6.7	8.8	7.0	10.6
Tosoh G8	%	5.6	4.5	6.7	8.8	7.0	10.6
Tosoh G7	%	5.6	4.5	6.7	9.9	7.9	11.9
Variant II	%	5.8	4.6	7.0	8.8	7.0	10.6
Primus Ultra2	%	5.4	4.3	6.5	9.0	7.2	10.8
Cobas Integra 400	%	6.0	4.8	7.2	9.6	7.7	11.5
*							

\*Add your analyzers values by establishing your own internal value assignment if not already listed

**Complementary NOD Liquid HbA1c Linearity is also available:**  
**DIABETES A1c LINEARITY 4 LEVEL**                      **Part No: HbL-G04041-100**

**NOVA-ONE Diagnostics<sup>®</sup>, LLC 22287 Mulholland Hwy Calabasas, CA. 91302**  
**818-348-1543 Fax 818-348-9696**  
**Email [nova-one@sbcglobal.net](mailto:nova-one@sbcglobal.net) / [www.nova-one.com](http://www.nova-one.com)**  
**(REV 06-01-2010)**